


Granton®



Knifemakers since 1601


GRANTON DISPOSABLE SCALPELS WITH RETRACTABLE BLADE

minimise the risk of sharp blade injuries and any subsequent infection to you and your staff. After use, the scalpel blade can be fully and automatically retracted into the handle with one hand, ensuring safe handling and disposal.

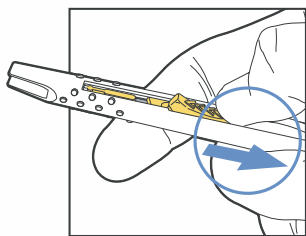


1

Pull back and hold safety cover to expose safety slide

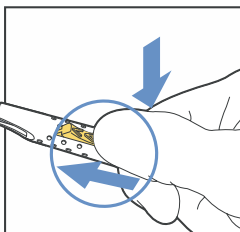


2

Push safety slide completely forward to extend blade

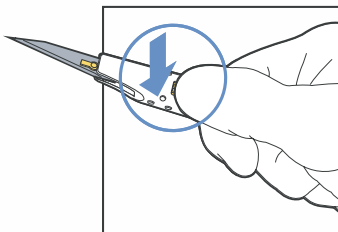

3

Push down front of safety slide to lock extended blade

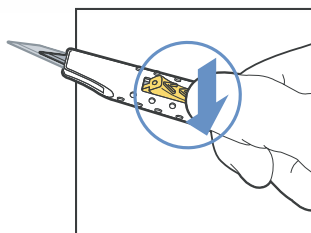

4

Push down back of safety slide to retract blade

- Double safety feature, prevents accidental blade extension
- Automatic retraction operation
- One-handed operation
- Enhanced safety during medical procedures
- Disposable
- Reliable
- Superior quality and balance
- Slip resistant finger grips
- Colour coded for easy recognition

RETRACTABLE SCALPEL	PRODUCT DESCRIPTION	QUANTITY
	Blade 10, Green	10/pack
	Blade 11, Yellow	10/pack
	Blade 15, Red	10/pack

SUPPLIED IN STERILE PACKS FOR SINGLE PATIENT USE ONLY

Granton Ragg Limited

Parkway Close, Parkway Industrial Estate, Sheffield, South Yorkshire S9 4WJ England

Tel: +44 (0)114 275 7290 Fax: +44 (0)114 272 7853

Email: info@granton.co.uk Website: www.granton.co.uk